

THD diagnostic and surgical line

Innovation and practicality for the coloproctologist



Document intended for Health Care Professionals only. Distribution of this document is restricted to the U.S.A. © 2019 THD S.p.A. All Rights Reserved

TABLE OF CONTENTS

CATEGORY	PRODUCT NAME	CODE	PAGE
	THD® Light-Scope ANO Beveled	800093	pg. 2
	THD [®] Light-Scope ANO Beveled STERILE	800113	pg. 2
	THD [®] Light-Scope ANO Truncated	800092	pg. 2
SELF-ILLUMINATING	THD [®] Light-Scope ANO Truncated STERILE	800112	pg. 2
ANOSCOPES	THD® N-ANO Beveled	800134	pg. 3
	THD [®] N-ANO Truncated	800040	pg. 3
	THD® N-ANO MAXI Beveled	800166	pg. 3
	THD [®] N-ANO MAXI Truncated	800165	pg. 3
	THD [®] Anoscope Pediatric Beveled	800021	pg. 4
	THD [®] Anoscope Pediatric Beveled STERILE	800012	pg. 4
	THD [®] Anoscope Pediatric Truncated	800020	pg. 4
	THD [®] Anoscope Pediatric Truncated STERILE	800011	pg. 4
	THD® Anoscope Medium Beveled	800024	pg. 5
TRADITIONAL	THD [®] Anoscope Medium Beveled STERILE	800015	pg. 5
ANOSCOPES	THD® Anoscope Medium Truncated	800023	pg. 5
	THD [®] Anoscope Medium Truncated STERILE	800014	pg. 5
	THD® Anoscope Maxi Beveled	800027	pg. 5
	THD® Anoscope Maxi Beveled STERILE	800018	pg. 5
	THD® Anoscope Maxi Truncated	800026	pg. 5
	THD [®] Anoscope Maxi Truncated STERILE	800017	pg. 5
	THD [®] Light-Scope Recto Mini	800098	pg. 7
	THD [®] Light-Scope Recto Mini STERILE	800117	pg. 7
	THD® Light-Scope Recto Mini with insufflation bulb	800100	pg. 7
SELF-ILLUMINATING	THD® Light-Scope Recto Mini with insufflation bulb STERILE	800119	pg. 7
RECTOSCOPES	THD [®] Light-Scope Recto MAXI	800097	pg. 7
	THD [®] Light-Scope Recto MAXI STERILE	800116	pg. 7
	THD [®] Light-Scope Recto MAXI with insufflation bulb	800099	pg. 7
	THD [®] Light-Scope Recto MAXI with insufflation bulb STERILE	800118	pg. 7
	THD® Surgy Mini LED	800108	pg. 9
SELF-ILLUMINATING	THD [®] Surgy Mini LED STERILE	800124	pg. 9
SURGICAL	THD® Surgy Maxi LED STERILE	800125	pg. 9
ANOSCOPES	THD [®] Light-Scope Procto HRA	800191	pg. 10
RUBBER BAND LIGATION	THD® Bandy	800183	pg. 12

THD DIAGNOSTIC AND SURGICAL LINE

Innovation and practicality for the coloproctologist

The THD diagnostic and surgical line is a complete range of THD disposable devices for diagnosing and treating anorectal diseases. THD devices are ideal for the diagnosis and treatment of numerous proctological diseases, such as hemorrhoids, polyps, condylomas, fistulas, neoplastic lesions, fissures and other lesions of the anorectal region. The continuous technological research and the work by a specialized pool of doctors, researchers and designers make THD devices the most innovative and versatile answer to the needs of specialists, guaranteeing the highest standards of safety, handling and precision of use.



BENEFITS

PRACTICAL

The self-illuminating devices do not need to be connected to a light source, allowing it to be used in any environment.

SAFE

The self-illuminating products are disposable, preventing any possibility of cross-contamination.

COST-EFFECTIVE

The reduced preparation time between one patient and the next allows more visits to be performed during a clinical session.

SELF-ILLUMINATING ANOSCOPES


THD[®] Light-Scope ANO LED THD[®] N-Ano THD[®] N-Ano Maxi

The THD self-illuminating anoscopes are single-use devices for inspecting the anal canal that are made of medical grade polycarbonate with a powerful built-in LED light.

The LED is activated by simply pulling out a tab, giving a clear and detailed view of the inspection area.



THD® Light-Scope ANO LED

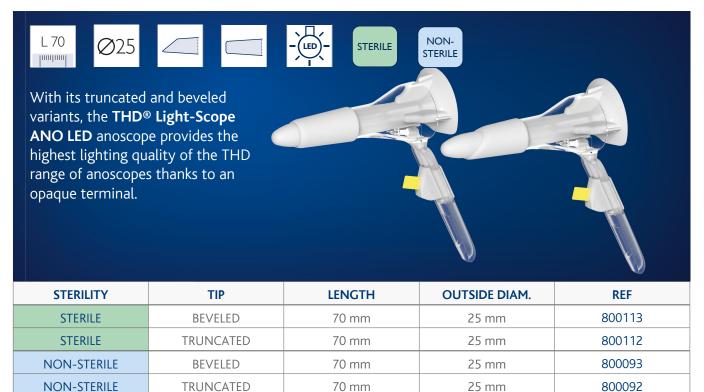


SELF-ILLUMINATING ANOSCOPES

THD® N-ANO

L91 The truncated and variants of the THI anoscope with LED best quality/price r range.	D® N-Ano) light offer the			
STERILITY	TIP	LENGTH	OUTSIDE DIAM.	REF
NON-STERILE	BEVELED	91 mm	21.7 mm	800134
NON-STERILE	TRUNCATED	91 mm	21.7 mm	800040

THD® N-ANO Maxi

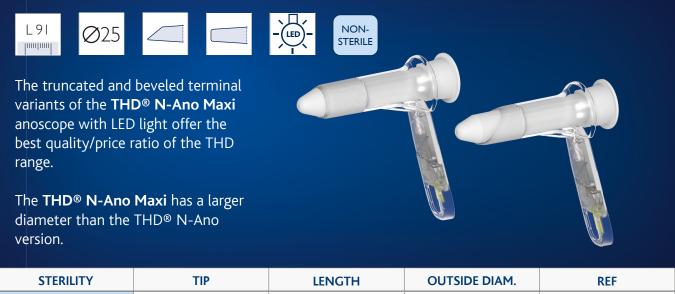

STERILITY	TIP	LENGTH	OUTSIDE DIAM.	REF
NON-STERILE	BEVELED	91 mm	25.8 mm	800166
NON-STERILE	TRUNCATED	91 mm	25.8 mm	800165

TRADITIONAL ANOSCOPES

THD[®] Anoscope Pediatric THD[®] Anoscope Medium THD[®] Anoscope Maxi

The line of traditional anoscopes are available in different lengths and diameters, and can be easily distinguished by the practical color coding of the handles. They are available in both beveled and truncated versions.

This high quality material is used to obtain a highly transparent, unbreakable, smooth product with a tapered and ergonomic shape that makes it lightweight and atraumatic.

The THD[®] Revolution optical fiber connection provides excellent lighting and visibility.

THD® Anoscope Pediatric

NON-STERILE

TRUNCATED



65 mm

800020

15 mm

TRADITIONAL ANOSCOPES

THD® Anoscope Medium




The **THD® Anoscope Medium**, with its truncated and beveled end variants, ensures safety and function for the surgeon.



STERILITY	TIP	LENGTH	OUTSIDE DIAM.	REF
STERILE	BEVELED	90 mm	22 mm	800015
STERILE	TRUNCATED	90 mm	22 mm	800014
NON-STERILE	BEVELED	90 mm	22 mm	800024
NON-STERILE	TRUNCATED	90 mm	22 mm	800023

THD® Anoscope Maxi

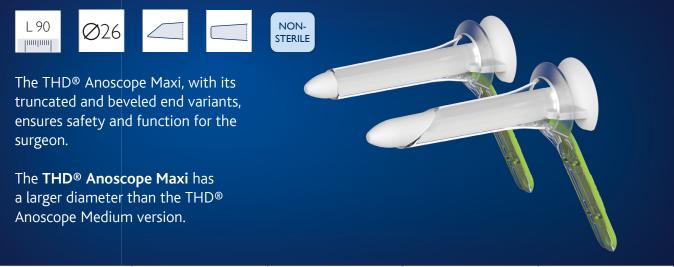

STERILITY	TIP	LENGTH	OUTSIDE DIAM.	REF
STERILE	BEVELED	90 mm	26 mm	800018
STERILE	TRUNCATED	90 mm	26 mm	800017
NON-STERILE	BEVELED	90 mm	26 mm	800027
NON-STERILE	TRUNCATED	90 mm	26 mm	800026

SELF-ILLUMINATING RECTOSCOPES

THD[®] Light-Scope Recto LED MINI & MAXI

The THD self-illuminating rectoscopes are single-use devices for inspecting the anorectal canal and the distal region of the sigmoid colon.

They are made of medical-grade polycarbonate and have a powerful built-in LED light.

The LED is activated by simply pulling out a tab, giving a clear and detailed view of the inspection area.

A magnifying glass magnifies the user's view of the inspected area by 4 times (4x).




THE THD SOLUTION

The sterile versions of the THD[®] Light-scope Recto LED rectoscopes are an effective solution for performing the "anastomosis air leak test" after an anorectal anastomosis.

The anastomosis air leak test needs to be performed at the end of resection surgery: this allows the surgeon to understand if the anastomosis is sealed or if another resection procedure is required.


The sterile version of the **THD® Light-Scope Recto LED** is recommended for use in the operating room, as it is pre-assembled and ready to use.

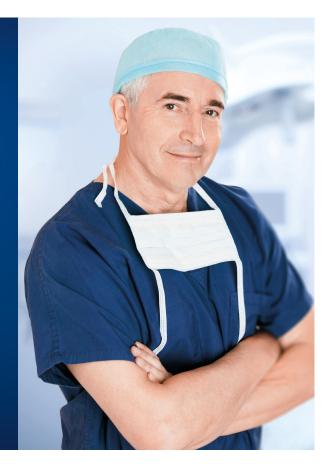

THD® Light-Scope Recto Mini LED



THD[®] Light-Scope Recto Maxi LED

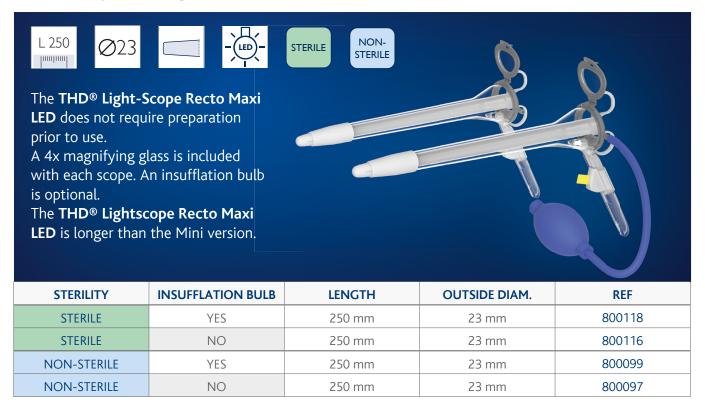

THD® SURGY

THD[®] Surgy

To aid the surgeon and to ensure patient safety, it is essential that the instruments used during transanal procedures are precise, manageable and ergonomic.

The THD surgical line offers THD[®] Surgy, the exclusive surgical retractor made of medical grade polycarbonate that is available in both Mini and Maxi versions.

THD[®] Surgy has a powerful built-in LED light that provides a clear and detailed view of the area being inspected and is activated by removing a pull tab.



THE THD SOLUTION

Anorectal surgical procedures have often been performed with inadequate lighting that has been accepted as a common limitation.

THD® Surgy provides intense light where it is needed, without annoying shadows or bright reflections from a metal retractor.

The ergonomics of using THD[®] Surgy are improved by the fact that there are no optical fiber cables and external light sources are not needed.

The materials used prevents any burns caused by accidental contact between the electrosurgical unit and a metal retractor.

This single-use system eliminates the risk of crosscontamination and the need for re-sterilization, which reduces the number of feasible procedures that can be performed.



THD® SURGY

THD® Surgy Mini LED



The compact and asymmetrical design of the THD® Surgy Mini LED allows easy access to the anal canal without sphincter trauma, ensuring good dilation and an excellent view of the operating field. The THD® Surgy Mini LED is recommended for patients with a narrow anal canal.



STERILITY	LENGTH	OPERATING WINDOW	REF
STERILE	122 mm	23x85 mm	800124
NON-STERILE	122 mm	23x85 mm	800108

THD[®] Surgy Maxi LED



THD® LIGHT-SCOPE PROCTO HRA

THD® Light-Scope Procto HRA

The THD[®] Light-Scope Procto HRA is a single-use device used for high resolution proctoscopy exams where inspection of the anal canal and margin are not viewed directly, but by magnification through a high resolution camera.

The camera connection adapter allows easy connecting/disconnecting and is equipped with a guard to prevent contamination of the camera.



THE THD SOLUTION

Early diagnosis and prevention play a fundamental role in managing anal dysplasia which leads to anal cancer.

The progression from anal dysplasia to anal cancer is slow allowing time for pre-cancerous lesions to be diagnosed and treated.

The THD[®] Light-Scope Procto HRA proctoscope defines a new treatment standard by allowing diagnostic biopsies and treatments to be performed under magnification.



THD® LIGHT-SCOPE PROCTO HRA

The large lateral openings allow the surgeon to insert biopsy forceps, gauze, pads and ablation tips inside the proctoscope without the need to remove the camera and avoids disruption of the magnified view.



THD® Light-Scope Procto HRA



The proctoscope is equipped with a built-in LED light source: the surface finish and the white colored terminal cone diffuse the light providing optimal visibility.

TIP

TRUNCATED

STERILITY

NON-STERILE



THD® BANDY

THD[®] Bandy

THD[®] Bandy is a single-use medical device for the treatment of low grade hemorrhoids.

THD Bandy is a suction rubber band ligator ready for immediate use that does not need to be cleaned and sterilized.

THD[®] Bandy includes the ligator and loading system with advancement cone. The surgeon positions the rubber band with one hand without the need of any assistance.



THE THD SOLUTION

THD[®] Bandy does not have the typical disadvantages of traditional systems.

The Bandy THD[®] ligator is made of a transparent plastic material, allowing the amount of aspirated tissue to be viewed directly. The single-use system eliminates the risk of cross-contamination and the need for sterilization, which reduces the time between one patient and the next.

The suction from the THD[®] Bandy ligator eliminates the need for forceps; the surgeon then has a free hand to maneuver the anoscope for easier insertion of the ligator. The THD[®] N-Ano Mini & Maxi single-use anoscopes, equipped with a built-in LED light and ergonomic shape, together with THD[®] Bandy, are the perfect combination for an optimal ligation.




THD® BANDY

THD® Bandy

L 150 STERILE The THD® Bandy is conr a suction system, which control the amount of as tissue. The rubber band is quick released by means of a c trigger.	is used to spirated	
STERILITY	DEVICE LENGTH	REF
NON-STERILE	150 mm	800183

THE BASIC STEPS

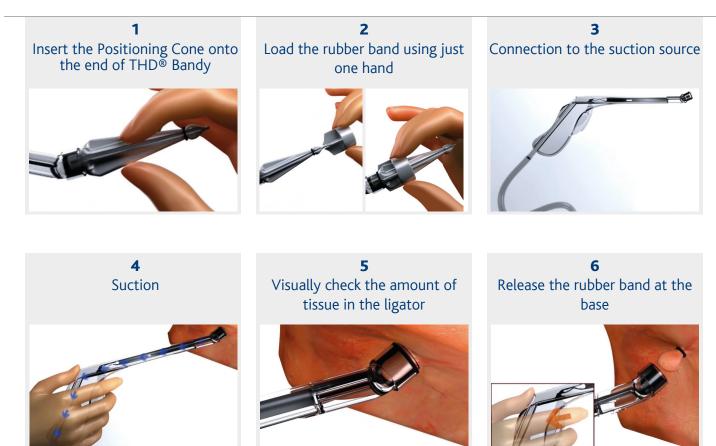

SPECIFICATIONS

CATEGORY	PRODUCT NAME	CODE
	THD® Light-Scope Beveled	800093
	THD [®] Light-Scope Beveled STERILE	800113
	THD® Light-Scope Truncated	800092
SELF-ILLUMINATING	THD [®] Light-Scope Truncated STERILE	800112
ANOSCOPES	THD® N-ANO Beveled	800134
	THD [®] N-ANO Truncated	800040
	THD® N-ANO MAXI Beveled	800166
	THD [®] N-ANO MAXI Truncated	800165
	THD [®] Anoscope Pediatric Beveled	800021
	THD [®] Anoscope Pediatric Beveled STERILE	800012
	THD [®] Anoscope Pediatric Truncated	800020
	THD [®] Anoscope Pediatric Truncated STERILE	800011
	THD [®] Anoscope Medium Beveled	800024
TRADITIONAL	THD [®] Anoscope Medium Beveled STERILE	800015
ANOSCOPES	THD [®] Anoscope Medium Truncated	800023
	THD [®] Anoscope Medium Truncated STERILE	800014
	THD [®] Anoscope Maxi Beveled	800027
	THD [®] Anoscope Maxi Beveled STERILE	800018
	THD [®] Anoscope Maxi Truncated	800026
	THD® Anoscope Maxi Truncated STERILE	800017
	THD [®] Light-Scope Recto Mini	800098
	THD [®] Light-Scope Recto Mini STERILE	800117
	THD® Light-Scope Recto Mini with insufflation bulb	800100
SELF-ILLUMINATING	THD® Light-Scope Recto Mini with insufflation bulb STERILE	800119
RECTOSCOPES	THD® Light-Scope Recto MAXI	800097
	THD [®] Light-Scope Recto MAXI STERILE	800116
	THD [®] Light-Scope Recto MAXI with insufflation bulb	800099
	THD® Light-Scope Recto MAXI with insufflation bulb STERILE	800118
	THD [®] Surgy Mini LED	800108
SELF-ILLUMINATING	THD [®] Surgy Mini LED STERILE	800124
SURGICAL ANOSCOPES	THD [®] Surgy Maxi LED STERILE	800125
	THD [®] Light-Scope Procto HRA	800191
RUBBER BAND LIGATION	THD® Bandy	800183

THD360780C - 2021/06/28 - © THD S.p.A.

THD America, Inc.

1731 SE Oralabor Road, Ankeny IA, 50021, USA **Ph.** +1 866-374-9442 **Fax** +1 813-626-0303 **www.thdlab.us**

