

THD SLIDE DOPPLER PROBE



<u>Use</u>

THD Slide Doppler Probe is intended to be used with THD Revolution and THD Evolution devices, for the treatment of haemorrhoidal disease, using the Transanal Haemorrhoidal Dearterialisation technique.

General description

THD Slide Doppler Probe shall be used with the THD Slide One and it has been designed for maximum sensitivity to identify blood vessels.

Its small size allows the physician easy handling and when used with the Slide One anoscope/proctoscope which has a port to lodge the Doppler, the operating field is completely unobstructed.

THD Slide Doppler Probe is available in both Reusable (for limited treatments) and Disposable versions.

The Disposable Probe is individually packaged and sterile.

The Disposable Probe may be sterilised by Ethylene Oxide (ETO).



Dimensional Features THD Reusable Doppler – THD Disposable Doppler **Body Dimension** 14x70x7 mm Cable Length 2000 mm

Product Datasheet

Information					
P/N	Pack	Sterilisation - Resterilisation			
800056	Not Sterile - Reusable	ETO, Plasma Gas (Hydrogen-peroxide, using Sterrad system) or Hyper acetic acid using Steris System (not System 1) Do not sterilize in an autoclave or at temperatures exceeding70°C.			
800063	Sterile - Disposable	Resterialization - Not admitted			

THD Slide						
P/N	Qty	Description				
800056	1	THD SLIDE DOPPLER PROBE (with 5 Shot) – Not Sterile				
800063	1	THD SLIDE DOPPLER PROBE – One, single shot, sterile doppler probe (Disposable)				
THD Doppler Probes are not intended for resale. A THD Doppler Probe is included in each THD Slide One Kit.						

Device or kit containing THD Doppler					
P/N Device	P/N Probe	Qty	Description		
800139	800063	1	THD REVOLUTION		
800068	800056	2	THD SLIDE ONE KIT w/5 Shot Doppler; 2 count Dopplers per kit		
800070-1		1	THD SLIDE ONE KIT; each kit contains one disposable doppler, sterile, in addition to other components of kit.		
800070-5	800063	5			
800070-10		10			

THD360109_US_Rev00_20180514_THD_Doppler_Probe.docx